

aerolet installation on a palma



hygiene beyond expectation

clos  mat™

installation process



1 With the seat lid in an upright position remove the seat from the Palma as follows.



2 Place your thumbs on the inside of both seat pins and apply pressure.



3 Remove seat and pillars complete and place to one side.



- 4** Remove the top cover.
Remove the central cap and retaining screw.



- 5** Pull the cover forward until it exposes the cistern, then lift away.



6



- 7** Fit the two supplied stainless steel replacement seat pin.
Make sure they are firmly seated, so that they depress evenly.



8 Picture showing location of anchor plates—in relation to the Aerolet frame.



9 Position both anchor plates into C-section frame behind porcelain pan.



10



11



12 Remove plastic cover to leg containing electrical connection.



13 These are held on via Velcro.



14



15 Locate the support arm that contains the switch operation and feed the wire down the locating leg.



16 Ensure the wire is not trapped when unit is in operation.



17 Route the wire through the frame and make connection to the pre-determined connector.



18 Slide the support arm into position and temporary tighten.

Adjust finally when unit is in position to suit clients requirements.



19 Position the Aerolet over the Palma and adjust the feet accordingly.



20 Make sure the cross bar from each Aerolet leg is sat firmly on the porcelain and level.



21 Attach the Aerolet to the anchor plates with the two stainless counter-sunk screws.

Caution must be taken to not trap any wires.



22 Aerolet secured in place.



23 Replace the cover ensuring that it slots in-between the anchor plates and the flat up-stand.



24 Adjust the height of the arms to client's requirements.



25 Re-screw the cover and replace the cap.



26 There are two versions of seat pins.
The shorter version is for the Hard Palma Seat.
The longer version is for the Soft Seat range.



27 All the Palma Seats are designed to fit the new Aerolets.

Remove the original seat pins from both seat and lid



28



29 Replace them with the correct stainless steel insert.



30 Locate the seat in the right position and secure.



31 Secure the seat to the Aerolet frame.



32 Electrically connect the lifter and raise to its highest position.

Re-locate all plastic covers as per the manufacturers instructions.



33 When the lifter is in the lowest position the seat pins must not be activated.



34 When the client sits on the lifter in the lowest position the seat pins will be activated.



Complete setup.

measurements

Measurements	Samoa-Vertical	Samoa-Tilting	Palma-Vertical	Palma-Tilting
width lift	720 mm	775 mm	720 mm	775 mm
length arm support	570 mm	680 mm	570 mm	680 mm
width arm support	670 mm	700 mm	670 mm	660 mm
depth frame lift	550 mm	560 mm	550 mm	560 mm
height frame lift	400 mm	400 mm	410 mm	410 mm
length foot cover	440 mm	405 mm	440 mm	405 mm
max. seat height	1000 mm (behind)	850 mm	1020 mm (behind)	850 mm
arm support height	680-1080 mm	630-890 mm	695-1095 mm	630-890 mm
height distance	430-850 mm	430-1000 mm	430- 850 mm	430-1000 mm

All measurements are based on 40 cm toilet-bowl height